


TENA Slip

Breathability for healthy skin

TENA Slip with ConfioAir™ is an all-in-one incontinence product, which helps to maintain healthy skin, providing breathability and leakage protection. ConfioAir™ allows skin to breathe and the new breathable backsheet and sides help maintain dryness for comfort and healthy skin.² TENA Slip now features new wide hook tapes which are easy to fasten and allow multiple adjustments to achieve a perfect fit. The fast-acting, highly absorbent core securely protects against leakage for comfort and dignity.

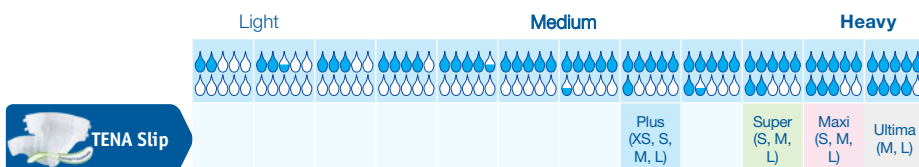

Product	Article Number	Hip Size	Working Absorbency	Total Packaging
TENA Slip Extra Small	713030	40 – 60cm	600mls	90 (3 x 30)
TENA Slip Plus Small	710133	50 – 80cm	700mls	90 (3 x 30)
TENA Slip Plus Medium	710630	70 – 110cm	900mls	90 (3 x 30)
TENA Slip Plus Large	710730	100 – 150cm	1000mls	90 (3 x 30)
TENA Slip Super Small	710030	50 – 80cm	750mls	90 (3 x 30)
TENA Slip Super Medium	711228	70 – 110cm	1000mls	84 (3 x 28)
TENA Slip Super Large	711428	100 – 150cm	1150mls	84 (3 x 28)
TENA Slip Maxi Small	710510	50 – 80cm	900mls	72 (3 x 24)
TENA Slip Maxi Medium	710924	70 – 110cm	1350mls	72 (3 x 24)
TENA Slip Maxi Large	711022	100 – 150cm	1650mls	66 (3 x 22)
TENA Slip Ultima Medium	710521	70 – 125cm	1500ml	63 (3 x 21)
TENA Slip Ultima Large	710621	90 – 145cm	1800ml	63 (3 x 21)

TENA Slip Extra Small is particularly suitable for children

88%
of carers see the skin health benefits¹

For the User: Curved leg elastics and security barriers provide increased comfort and greater protection against urinary and faecal incontinence.

For the Carer: Easy Hook Tape fastening allows the fixation tapes to be anchored into position and repositioned to ensure a snug fit. Colour-coded for appropriate product selection. Excellent leakage protection reduces frequency of pad changes.

For the Purchaser: Reduced product usage and continence promotion saves time and money.

Feature

Benefit

- **FeelDry™ Layer** For quick absorption of liquids for maximum leakage security; enhanced dryness for improved skin care
- **Odour Neutralizer** To reduce any unpleasant smells
- **Soft Textile Backing** Ensures enhanced comfort and dryness for improved skincare
- **Wetness Indicator** To advise when the pad needs changing
- **Dermatologically tested** All material is dermatologically tested to ensure skin health
- **Gentle to the skin** The soft and gentle textile-like material is gentle to the skin
- **ConfioAir™ allows skin to breathe** New breathable backsheet and sides maintain dryness for comfort and healthy skin¹
- **New wide hook tapes** Easy to fasten and adjust multiple times to achieve a perfect fit
- **Fast-acting, highly absorbent core** Securely protects against leakage for comfort and dignity



ConfioAir™ is now available in Medium and Large sizes across all absorbencies.

Remaining sizes will be available soon. 1. Product testing with 242 carers in three countries:

88% confirm that TENA Slip with ConfioAir™ is more airy and helps to protect skin health. 2.

Zhai H, Maibach HI. Occlusion vs. skin barrier function. Skin Res Technol. 2002 Feb;8(1):1-6.

How to fit TENA Slip - Standing & Lying




With the user standing, place the pad between the legs from front to back.



Fit both the bottom hook tapes, one at a time, angled slightly upwards.



Then fit both top tapes, one at a time angled slightly downwards. Check for comfort and fit. The pad allows for multiple refastening and adjustment of the hook tapes.

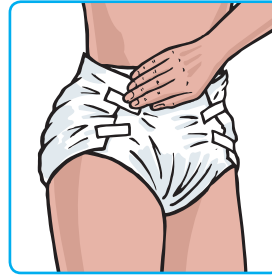

Gently press and smooth the hook tapes on to the textile-like back-sheet.



Give a gentle pull to anchor the hook tapes into the fibres of the back-sheet. This ensures the secure fixation.



With the user lying on their side, slide the pad between the legs from front to back.



Smooth out the pad.



Fit both the bottom hook tapes, one at a time, angled slightly upwards and then fit both top hook tapes, one at a time angled slightly downwards. Check for comfort and fit. The pad allows for multiple refastening and adjustment of the hook tapes.



Gently press and smooth the hook tapes on to the textile-like back-sheet.



Give a gentle pull to anchor the hook tapes into the fibres of the back-sheet. This ensures the secure fixation.

TENA Slip dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit from front to back
- ✓ Seal bottom tabs pointing upwards and top tabs pointing downwards
- ✓ Ensure standing gathers around leg openings are pulled up before fitting
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively
- ✗ Never 'twist' the product
- ✗ Never shake the product in order to open

People who bought TENA Slip also bought...



TENA U-test

TENA U-test offers a solution for easy and comfortable detection of urinary tract infections. Individuals are spared unnecessary discomfort and maintain dignity, while lowering the workload for carers.



TENA Shampoo Cap

The new TENA Shampoo Cap is pre-moistened with a non-rinse hair washing formula. Ideal in situations where normal hair washing is challenging or not possible.



TENA Wet Wash Glove

TENA Wet Wash Glove enables a comfortable and convenient cleansing procedure. Impregnated with a non-rinse cleansing lotion.

SCA Hygiene Support



Email: hcmarketing@sca.com
www.TENA.co.uk



TENA Education Academy



TENA Nursing Support